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Switzerland ☎ +41 448850634

EN Operating Instructions

Lunos® Polishing Paste Two in One Neutral / Orange / Mint with fluoride

Two in One Polishing Paste is available in the following variants:

Mint (REF: CPZ730A1901), contains 1500 ppm sodium fluoride & peppermint oil

Orange (REF: CPZ720A1901), contains limonene

Neutral (REF: CPZ710A1901), without fluoride, colouring or flavouring agents

Notice

Two in One Polishing Paste is a medical product. This product is only intended for professional dental applications and must only be used by trained specialists.

Two in One Polishing Paste is a high-quality polishing paste for use in dental prophylaxis. Two in One Polishing Paste removes discolouration and plaque extremely gently. Two in One Polishing Paste works on the innovative principle of „self-reduction“. In other words, the initially larger rod-shaped polishing particles are reduced in size and broken down during the polishing process. As a result, in a single process first the plaque is removed quickly, gently and thoroughly in the first 10 seconds of polishing, and immediately afterwards the tooth surfaces are gently smoothed and polished. Two in One Polishing Paste has an RDA of 28*.

Indications

The preparation is suitable for cleaning fissure sealing. Here, the fluoride-free variants „Orange“ or „Neutral“ are to be used preferably.

Contraindications

Two in One Polishing Pastes are contraindicated for patients with known allergies to the ingredients used in the pastes. (E.g. limonene; peppermint oil)

Warnings

Contact with the eyes may cause irritation. If this happens rinse with plenty of water.

Precautionary measures

When working with Two in One Polishing Paste, always use appropriate protective equipment: disposable gloves, face mask and protective goggles.

The Mint Polishing Paste with fluoride does not replace a dedicated fluoride therapy.

Application

Take a sufficient amount of polishing paste and fill it into an application ring, dappen glass or similar, suitable vessel. Use a suitable polishing instrument to take up polishing paste and start the polishing process. Please note that the self-reducing effect of the paste starts after around approx. 10 seconds. Keep polishing until the polishing particles have broken down into fine particles, so that a smoothing effect can be achieved once the plaque has been removed. Please note that there must always be enough of the polishing paste on the polishing instrument and on the tooth to prevent excessive build-up of heat on the tooth. For this reason, please also make sure that the maximum recommended rpm setting recommended by the polishing instrument manufacturer is not exceeded. Observe the operating instructions for use.

Storage and shelf life

Please ensure that you carefully seal the tubes again after removal of the paste to prevent the paste from drying out.

Do not store above 25°C and protect against direct sunlight. The shelf life of the product is 30 months.

Composition

Two in One Mint: Silicates, glycerin, aqua dest., hydroxylapatite, acidity regulator, preservatives (phenoxyethanol, parabens), thickening agent, emulsifying agent, sodium fluoride (1500 ppm), colouring agents (C.I. 42090, C.I. 47005, C.I. 77891), peppermint oil

Two in One Orange: Silicates, glycerin, aqua dest., hydroxylapatite, acidity regulator, preservatives (phenoxyethanol, parabens), thickening agent, emulsifying agent, colouring agents (C.I. 47005, C.I. 77891), orange flavouring (limonene)

Two in One Neutral: Silicates, glycerin, aqua dest., hydroxylapatite, acidity regulator, preservatives (phenoxyethanol, parabens), thickening agent, emulsifying agent, colouring agents (C.I. 77891)

* Method according to Stookey/Schemehorn with reference substance pumice stone for prophylaxis pastes [J Dent Res, 1979 58(2): p.588-592], relative RDA value in comparison to product „C“ = RDA 27
Measurement results: Product „C“ = 255 / Two in One = 260

DÜRR DENTAL (Products) UK Ltd.

14 Linnell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom

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DÜRR DENTAL ITALIA S.r.l.
orochemie GmbH + Co. KG