DE Lunos® Polierpaste Super Soft
Orange / Neutral

EN Lunos® Prophy Paste Super Soft
Orange / Neutral

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EN Instruction for use

Lunos® Prophy Paste Super Soft Orange / Neutral

The following versions of Polishing Paste Super Soft are available:

Orange (REF: CPZ750A0901), contains hydroxylapatite, contains limonene
Neutral (REF: CPZ740A0901), contains hydroxylapatite, contains no colorants or flavouring agents

Notice

Polishing Paste Super Soft is a medical device. This product is only intended for professional dental applications and must only be used by trained specialists!
Polishing Paste Super Soft is a high-quality polishing paste for use in dental prophylaxis. Polishing Paste Super Soft removes soft plaque extremely gently and polishes sensitive surfaces. Polishing Paste Super Soft has an RDA of < 5*.

Indications

Polishing Paste Super Soft is used for dental prophylaxis by trained specialists. The preparation is suitable for cleaning fissure sealing. Polishing Paste Super Soft is particularly well-suited for the cleaning of sensitive surfaces, such as implants and milk tooth surfaces.

Contraindications

Polishing Pastes Super Soft are contraindicated for patients with known allergies to the ingredients of the pastes (e.g. limonene).

Warnings

Contact with the eyes may cause irritation; if this occurs, rinse with copious amounts of water.

Precautionary measures

Working with Polishing Paste Super Soft, make sure to use appropriate protective equipment: disposable gloves, face mask and protective goggles

Polishing Pastes Super Soft may be used by suitably trained professionals only.

Application

Take a sufficient amount of polishing paste and fill it into an application ring, dappen dish or similar, suitable vessel. Use a suitable polishing instrument to take up polishing paste and start the polishing process. Please note that there must always be enough polishing paste on the polishing instrument and on the tooth to prevent excessive build-up of heat on the tooth. For this reason, please also make sure that the maximum speed recommended by the manufacturer of the polishing instrument is not exceeded. Comply with the operating instructions of the manufacturer.

Storage and shelf life

Please ensure that you carefully seal the tubes again after removal of the paste to prevent the paste from drying out.

Do not store above 25°C and protect against direct sunlight. The shelf life of the product is 30 months.

Composition

Polishing Paste Super Soft Orange

Hydroxylapatite, glycerol, aqua dist., thickener, preservative (phenoxyethanol, parabene), emulsifying agent, orange flavouring (limonene), colorant (C.I. 15985, C.I. 16255)

Polishing Paste Super Soft Neutral

Hydroxylapatite, glycerol, aqua dist., thickener, preservative (phenoxyethanol, parabene), emulsifying agent

* Method of Stookey/Schemehorn using pumice as a reference substance for prophylaxis pastes [J Dent Res, 1979 58(2): p.588-592], relative RDA value as compared to product "P" = RDA 7, Measuring results: Product "P" = 126 / Super Soft = 15

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