**DE** Lunos® Prophylaxepulver Gentle Clean  
Neutral / Orange / Spearmint

**EN** Lunos® Prophylaxis Powder Gentle Clean  
Neutral / Orange / Spearmint

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## EN Operating Instructions

### Lunos® Prophylaxis Powder Gentle Clean

The following versions of Prophylaxis Powder Gentle Clean are available:

**Spearmint** (REF: CPZ630A2250), contains limonene  
**Orange** (REF: CPZ620A2250), contains limonene  
**Neutral** (REF: CPZ610A2250), contains no flavouring agents

**Notice:** Prophylaxis Powder Gentle Clean is a medical device. This product is only intended for professional dental applications and must only be used by trained specialists!

„Prophylaxis Powder Gentle Clean“ is designed for supragingival application on the surface of the tooth using commercial powder jet devices. Please note the operating instructions of the powder jet device before use of the prophylaxis powder.

#### Application

This product is recommended for:

- Supragingival cleaning of the tooth surface
  - Removal of discoloration and plaque
- Preparation and cleaning of the tooth surface before:
  - Adhesive attachment of orthodontic brackets
  - Application of fluoride and bleaching preparations
  - Insertion of composite restorations, fixed dentures, e.g. inlays, onlays, crowns, veneers etc.
  - Application of sealing materials

#### Contraindication

Patients suffering from chronic airway diseases (e.g. bronchitis or asthma) must not be treated with a powder-water jet device since the powder present in the aerosol can lead to difficult breathing. Likewise, the product should not be used on non-intact enamel and hypomineralised tooth enamel and „white spots“.

In certain cases, flavouring agents present in the powder may cause skin irritation or allergic reaction. For this reason, patients with known allergies to flavouring agents should be treated with „Prophylaxis Powder Gentle Clean Neutral“.

Please make sure not to direct the powder jet device directly at fixed dental restorations and fillings as this may damage the restorations.

Use „Prophylaxis Powder Gentle Clean“ in supragingival locations exclusively. Please make sure to use „Prophylaxis Powder Gentle Clean“ in powder jet devices for supragingival application exclusively.

#### Preparation

- Please remove all residual powder from the powder chamber of the powder jet device before refilling it with Prophylaxis Powder Gentle Clean
- Shake the closed bottle before removing any Prophylaxis Powder Gentle Clean. Please remove the seal on the bottle

before removing the powder. For this purpose, take the lid off the bottle, pull off the seal and then screw the lid back on the bottle.

- Before filling the powder prophylaxis powder chamber, make sure that the chamber is completely dry. There must be no residual moisture in the chamber, e.g. from reprocessing measures.
- Filling the powder into the powder chamber, you need to make sure not to exceed the maximum filling level of the powder chamber.
- Fold the „rocker“ all the way up or unscrew the lid from the bottle. To minimise the amount of dust generated when you fill the chamber, it is recommended to use the „rocker“ for this purpose and to slightly compress the bottle.

#### Recommendations for treatment

Patient and operator need to wear suitable eye protection, e.g. protective goggles. The operator needs to wear suitable protective equipment in the form of mouth and nose protection and suitable protective gloves.

Use a suitable spray mist aspiration cannula, e.g. the Dürr Dental Prophylaxis Canula.

Do not direct the spray nozzle at the gingiva during the treatment. Direct exposure of the soft tissue may damage the gingiva. The powder jet nozzle must not be aimed directly into the gingival pocket as there is a risk of emphysema arising in the tissue and/or of embolism from air being introduced into blood circulation.

To protect sensitive skin, please apply Vaseline to the patient's lips before commencing the treatment.

After completion of treatment, clean the patient's face with a moist disposable cloth.

This prophylaxis powder cannot be used remove tartar.

#### General Information

This prophylaxis powder does not have a cariogenic effect.

Please make sure to always remove any residual powder from working surfaces, front surfaces of cabinets or treatment units using a moist cloth.

#### Notes for the patient

Please advise your patient that the ingestion of food or beverages (tea, coffee or other food items) within the first two to three hours after completion of the treatment can cause renewed discoloration of the teeth.

#### Storage and shelf life

Store in a well-ventilated place at a temperature not above 25°C. The shelf life of the product is 30 months.

#### Composition

**Prophylaxis Powder Gentle Clean Neutral:** Trehalose, excipients

**Prophylaxis Powder Gentle Clean Orange:** Trehalose, excipients, orange flavouring (limonene)

**Prophylaxis Powder Gentle Clean Spearmint:** Trehalose, excipients, spearmint flavouring (limonene)

DÜRR DENTAL (Products) UK Ltd.

14 Linnell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom

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